


ADD-Vantage[®] System

As Easy as 1, 2, 3

1 Assemble

Use Aseptic Technique

Swing the pull ring over the top of the vial and pull down far enough to start the opening. Then pull straight up to remove the cap. Avoid touching the rubber stopper and vial threads.

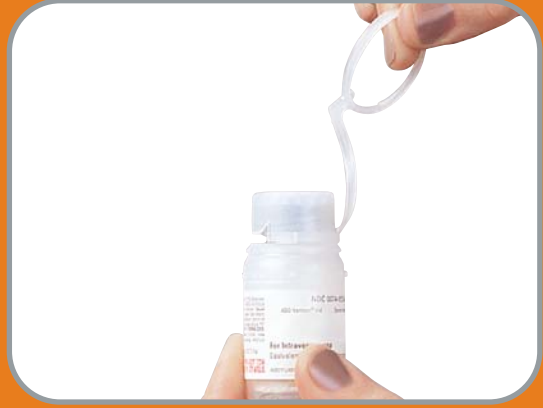

Hold diluent container and gently grasp the tab on the pull ring. Pull up to break the tie membrane. Pull back to remove the cover. Avoid touching the inside of the vial port.



Screw the vial into the vial port until it will go no further. Recheck the vial to assure that it is tight. Label appropriately.



2 Activate

Pull Plug/Stopper to Mix Drug with Diluent

Hold the vial as shown. Push the drug vial down into container and grasp the inner cap of the vial through the walls of the container.



Pull the inner plug from the drug vial: allow drug to fall into diluent container for fast mixing. Do not force stopper by pushing on one side of inner cap at a time.



Verify that the plug and rubber stopper have been removed from the vial. The floating stopper is an indication that the system has been activated.



3 Mix and Administer

Within Specified Time

Mix container contents thoroughly to assure complete dissolution. Look through bottom of vial to verify complete mixing. Check for leaks by squeezing container firmly. If leaks are found, discard unit.



Pull up hanger on the vial.



Remove the white administration port cover and spike (pierce) the container with the piercing pin. Administer within the specified time.

